

DENTAMAT 3000 X-ray Developer and Fixer

Beschreibung :

DENTAMAT 3000 is a dental film-developing set (developer and fixer) for developing of dental films in immersion developing machines (Clarimat Super S II und Super S III, Girardelli X-20 und X-24 a.o.).

The concentrates secure optimal processing and ability for archiving of the dental films .

The manual processing in dish or tank is also possible .

Developer- and fixer concentrates are one - part .

Mix Instructions :

preparation fixing bath :

1. Pour out fixer concentrate into the fixing tank .
2. Fill the empty bottle with water (20 - 23 °C) and fill up the fixing tank to the final volume .
3. Stir the ready to use solution thoroughly .

preparation developer :

1. Pour out developer concentrate into the developing tank .
2. Fill the empty bottle with water (20 - 23 °C) and fill up the developing tank to the final volume .
3. Stir the ready to use solution thoroughly .

Manual Processing :

Preparation of ready to use developer and fixer solution from concentrate :

$$3,0 \text{ l concentrate} + 6,0 \text{ l water} = 9,0 \text{ l developer or fixer}$$

(dilution rate 1 + 2)

Replenishment :

When required fill up the respective tank of the immersion developing machine with water to the final volume - at least once a week !

Processing Conditions :

Attention ! Because of the X - ray acceptance inspection the temperature of the processing bathes should be **about 24 °C** .

DENTAMAT 3000 X-ray Developer and Fixer

Durability and Utilization Capacity of the Bathes :

With a machine filling the developing of dental films is possible about 3 weeks .
 Fresh preparation is necessary after 3 weeks at the latest - also at low throughput .
 At the manual processing the ready for use bathes can be used until the activity has gone out .

Watering Tank :

To get perfect X - ray films with ability for archiving, the washing water has to be changed twice a week .

Assortment :

Order Number	Product Name	Mix Size
10096	DENTAMAT 3000	1 / 1 x 3,0 l conc. developer and fixer

Storage and Handling :

- Until the preparation **DENTAMAT 3000** has to be kept in the closed original package and stored at temperatures of 10 to 25 ° C .
- All users of photographic processing solutions should exercise the greatest care to avoid the chemicals contacting the skin or eyes . Always wear solution resistant gloves .
- The disposal of used photographic processing solutions has to be occurred in accordance with the legal regulations .