

T 325 X-ray Tank Fixer

Description :

- concentrate - fixer for tank processing of medical and technical X - ray films;
- use in conjunction with the X - ray tank developer T 31 preferably;
- liquid concentrate;
- one - part.

Processing Conditions :

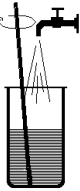
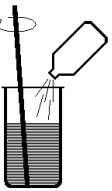
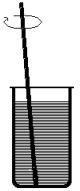
fixing temperature :	18 °C	fixing time :	5 - 6 min.
	20 °C		4 - 5 min.
	22 °C		3 - 4 min.
	24 °C		2 - 3 min.

Replenishment :

- For processing of 1 m² X - ray film 500 - 600 ml tank solution are necessary at standard conditions .

Mix Instructions :

preparation X - ray fixer :

Water (25 - 30 °C)	T 325 - Concentrate	= Tank or Replenisher solution
	 (Add under permanent stirring)	
20 l	5 l	25 l
4 l	1 l	5 l

T 325 X-ray Tank Fixer

Assortment :

Order Number	Product Name	Mix Size for	Capacity (m ²)
10080	T 325	2 x 25,0 l	80 - 100
10041	T 325	10 x 5,0 l	80 - 100

Storage and Handling :

- Until the preparation T 325 has to be kept in the closed original package and stored at temperatures of 10 to 25 ° C .
- All users of photographic processing solutions should exercise the greatest care to avoid the chemicals contacting the skin or eyes . Always wear solution resistant gloves .
- The disposal of used photographic processing solutions has to be occurred in accordance with the legal regulations .