

UNIMATIC-D X-ray Developer for Table Processors

Description :

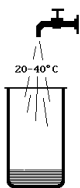
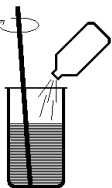
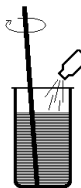
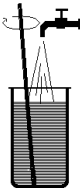
- X - ray developer for all customary table processors and machines with small tank volume;
- Product secures middle up to high contrast with good stability.
- especially suitable for processing in small tank - and table processors like Agfa Gevamatric 60 and Agfa Curix Compact;
- liquid concentrate;
- two - part;

Replenishment :

- For processing of 1 m² X-ray film 400 to 600 ml replenisher are necessary at standard conditions.
- A higher replenishment rate is necessary at longer standstill time without processing or at low utilization .

Mix Instructions :

preparation X - ray developer replenisher :

Water (25 - 30 °C)	Part 1	Part 2	fill up with Water to
			
1,2 l	1,0 l	0,25 l	2,5 l

- First dissolve part 1 completely in water, afterwards add part 2 under permanent stirring ;
- Use of starter concentrate is not necessary .

UNIMATIC-D X-ray Developer for Table Processors

Assortment :

Order Number	Product Name	Mix Size for	Concentrate	Capacity m ²
10092	UNIMATIC-D	6 x 2,5 l	Teil 1 6 x 1,0 l Teil 2 6 x 0,25l	25 - 35

Storage and Handling :

- Until the preparation the concentrate has to be kept in the closed original package and stored at temperatures of 10 to 25 °C .
- **UNIMATIC-D** contains components which can cause irritations in case of contact with the skin.
- In case of contact with the eyes it has to be rinsed thoroughly with plenty of running cold water by opened eyelid and a medical doctor has to be consulted.