

UNIMATIC-F X-ray Fixer for Table Processors

Description :

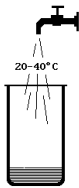
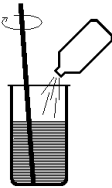
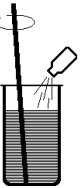
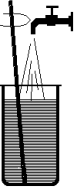
- X - ray fixer for all customary table processors and machines with small tank volume;
- especially suitable for processing in small tank - and table processors like Agfa Gevamat 60 and Agfa Curix Compact;
- Good hardening properties prevent mechanical damages of the passing films reliably .
- Special fixer components secure the fix out also at fast going machines .
- liquid concentrate;
- two - part;

Replenishment :

- For processing of 1 m² X - ray film 650 to 800 ml UNIMATIC-F are necessary at standard conditions

Mix Instructions :

preparation X - ray fixer :

Water (25 - 30 °C)	Part 1	Part 2	fill up with Water to
			
1,9 l	0,5 l	0,1 l	2,5 l

- First dissolve part 1 completely in water, afterwards add part 2 under permanent stirring .
- Direct mix of the concentrate parts without water has to be avoided absolutely .

UNIMATIC-F X-ray Fixer for Table Processors

Assortment :

Order Number	Product Name	Mix Size for	Concentrate	Capacity m ²
10093	UNIMATIC-F	6 x 2,5 l	Teil 1 6 x 0,5 l Teil 2 6 x 0,1 l	18 - 23

Storage and Handling :

- Until the preparation the concentrate has to be kept in the closed original package and stored at temperatures of 10 to 25 °C .
- **UNIMATIC-F** contains components which can cause irritations in case of contact with the skin.
- In case of contact with the eyes it has to be rinsed thoroughly with plenty of running cold water by opened eyelid and a medical doctor has to be consulted.