

## Xroll-D X-ray Developer for Roller Machines

### Description :

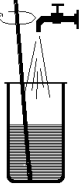
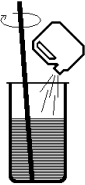
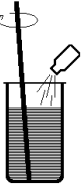
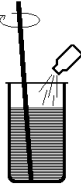
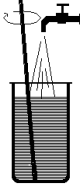
- X - ray developer for all customary developing machines with roller transport and for spray developing machines ;
- high utilization of sensitivity ;
- optimal sensitivity - fog - proportion at high light contrast ;
- especially recommended for machines with high and regular film utilization ;
- liquid concentrate ;
- three - part.

### Replenishment :

- For processing of 1 m<sup>2</sup> X - ray film **400 to 600 ml** replenisher are necessary at standard conditions.
- Generally the replenishment occurs automatically by addition of the solution in dependence of the processing film.

### Mix Instructions :

preparation X - ray developer replenisher :

Water (25-30 °C)	Part 1	Part 2	Part 3	fill up with Water to
				
14 l	5 l	0,5 l	0,5 l	20 l

- First dissolve part 1 completely in water, afterwards add part 2 under permanent stirring and then part 3 in the same way .

preparation X - ray developer working solution :

20 ml Uni - Starter per litre contents of tank has to be added the prepared replenisher .

**Xroll-D X-ray Developer for Roller Machines**

**Assortment :**

Order Number	Product Name	Mix Size for	Concentrate	Capacity m <sup>2</sup>
10111	XROLL-D	2 x 20 l	Part 1 2 x 5,0 l Part 2 2 x 0,5 l Part 3 2 x 0,5 l	65 -100

**Storage and Handling :**

- Until the preparation the concentrate has to be kept in the closed original package and stored at temperatures of 10 to 25 °C .
- **Xroll-D** contains components which can cause irritations in case of contact with the skin.
- In case of contact with the eyes it has to be rinsed thoroughly with plenty of running cold water by opened eyelid and a medical doctor has to be consulted.