

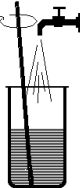
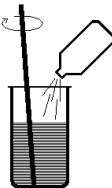
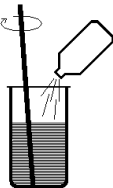
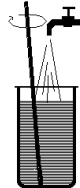
DENTAROLL X-ray Developer and Fixer

Description:

DENTAROLL is a dental film-developing set (developer and fixer) for developing of dental films, special in roller-developing machines (Dürr AC 1330, AC 1416, AC 245 L and XR 24, Philips Rollomat 810 and 820, Gendex Clarimat GX-300, Velopex (preparation 1 + 3), Siemens P 10).
The concentrates secure optimal processing and ability for archiving of the dental films .
Developer – and fixer concentrates are two – part .

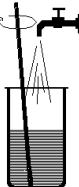
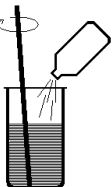
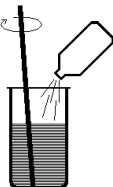
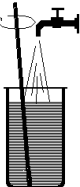
Mix Instructions :

preparation fixing bath :

Water (20 – 23 °C)	Part A	Part B	fill up with Water to
			
<p>about 3 l</p>	<p>Fill empty bottles with water and fill up the fixing tank to the final volume . Stir the ready to use solution thoroughly !</p>		<p>5 l</p>

DENTAROLL X-ray Developer and Fixer

preparation developer :

Water (20 - 23 °C)	Part A	Part B	fill up with Water to
			
<p>about 3 l</p>	<p>Fill empty bottles with water and fill up the developing tank to the final volume . Stir the ready to use solution thoroughly !</p>		<p>5 l</p>

DENTAROLL X-ray Developer and Fixer

Replenishment :

When required fill up the respective tank with water to the final volume – at least once a week !

Processing Conditions :

Attention ! Because of the X – ray acceptance inspection the temperature of the processing bathes should be about 24 °C .

Durability and Utilization Capacity of the Bathes :

With a machine filling the developing of dental films is possible about 4 weeks . Fresh preparation is necessary after about 3 month – also at low throughput .

Watering Tank :

To get perfect X – ray films with ability for archiving the films have to be treated with fresh water all the time .The water is supplied automatically only at the developing and flow away afterwards .

Assortment :

Order Number	Product Name	Mix Size For
10090	DENTAROLL	2 / 2 x 5 l ready to use developer and fixer

DENTAROLL X-ray Developer and Fixer

Storage and Handling :

- Until the preparation DENTAROLL has to be kept in the closed original package and stored at temperatures of 10 to 25 ° C .
- All users of photographic processing solutions should exercise the greatest care to avoid the chemicals contacting the skin or eyes . Always wear solution resistant gloves .
- The disposal of used photographic processing solutions has to be occurred in accordance with the legal regulations .