---

## ISOCLEAN

---

### Liquid processor cleaner for Micrography, X-ray, B&W Photo Flüssiger Maschinenreiniger für Mikrografie, Röntgen, S/W Photo

**Specially designed** for the cleaning of tanks with **hydroquinone-free developers** - suitable also for any other tanks in monochrome processing machines.

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## APPLICATION

---

- Drain the developer tank. Remove chemistry filter.
- Fill the tank with **warm water** and circulate for ca. 3-5 minutes.
- Drain the tank again and close the drain tap.
- The rack may remain in the tank.
- Prepare ISOCLEAN in a bucket or in a mixing container, add it to the developer tank.
- Switch on the machine, resp. the circulation,
- Let ISOCLEAN work for approx. 30 minutes.
- Drain the tank, close the drain tap. Dispose of used cleaner solution.
- **Rinsing:** Fill the tank with **warm water**, circulate for ca. 3-5 minutes. Drain the tank.
- **Repeat the rinsing procedure 2 x!** Insert new filter.

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## STORAGE · LAGERUNG

---

ISOCLEAN **concentrates** should be stored dry, frost-proof and out of the reach from children. The maximum temperature range is between 5°C and 25°C.

Storage temperatures between 10°C and 20°C are ideal.

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## DISPOSAL · ENTSORGUNG

---

**Photo chemicals** - concentrates or used solutions - must not be discharged into public sewerage systems. Photo chemicals, that are no longer needed or can no longer be used must be disposed of by licensed waste treatment companies in accordance with local legal restrictions.

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