

T 325

MANUAL X-RAY FILM FIXER CONCENTRATE

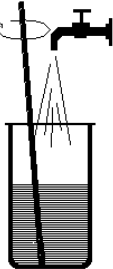
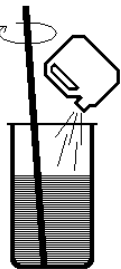
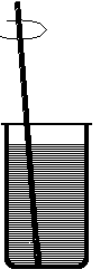
Product Description:

T 325 is an x-ray film fixer concentrate, suitable for manual processing of medical and also technical x-ray films. It provides high speed fixing performance, preferably in combination with T 32 Developer excellent results can be achieved. T 325 is a single-part product.

Processing Conditions:

Fixing Temperature	18 °C	Fixing Time	5 - 6 min
	20 °C		4 - 5 min
	22 °C		3 - 4 min
	24 °C		2 - 3 min

Mix Instruction Tank Solution & Replenisher:

Order #	to make	Water 20°C	Concentrate			Solution
						
10080	2 x 25 L	20 L	5,0 L			25 L
10041	4 x 5 L	4 L	1,0 L			5 L

Dissolve concentrate into water while stirring continuously until completely dissolved.

Storage & Handling:

Until preparation of replenisher / tank solution all parts should be kept in originally closed and sealed containers / bottles to avoid oxydation. Recommended storage temperature is 10 - 25 °C. Please avoid storage at < 5 °C as well as high humidity and direct sunlight.

Hazardous goods information:

Product contains substances that can cause irritations and chemical burns of eyes and skin. For storage, transport and handling see actual Material Safety Data Sheet (MSDS). Product can be transported without any restriction according ADR / IMDG.